TEOSYAL Global Action 30G

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Instruction leaflet

Description

Teosyal 30G Global Action is a transparent, non pyrogenic, visco-elastic gel of cross-linked hyaluronic acid, of non animal origin. Each box contains one or two pre-filled syringes of Teosyal 30G Global Action. For each syringe, the box contains one or two sterile 30G1/2" needles and two tracability labels (one to be given to the patient, and one to be kept by the doctor in the patient's file). The volume of each syringe is shown on the packaging carton as well as on each syringe.

Composition: Cross-linked hyaluronic acid 25 mg Phosphate buffer pH 7,3 qsp ad 1 ml

Indications

The therapeutic purposes of Teosyal® products are: - to modify the anatomy of aged skin: restoration of volumes, filling of skin depression and wrinkle, restoration of skin hydration, - reconstructive surgery: filling of depressions due to scars, reconstruction of volumes lost by lipotrophy. Teosyal 30G Global Action is indicated for filling facial wrinkles, creases like slight or moderate folds, wrinkles around the mouth and glabella wrinkles.

Mode of action

Teosyal 30G Global Action is injected into the mid dermis at the site of the cutaneous depression to be corrected. Teosyal 30G Global Action creates a volume that corrects wrinkles. Teosyal 30G Global Action is biodegradable and is slowly resorbed over time. In general one, but sometimes two treatment sessions are necessary, depending upon the depth of the wrinkles in need of treatment, to obtain an optimal degree of correction. Periodic touch-up sessions then enable the sought after degree of correction to be maintained.

Contra-indications

Teosyal 30G Global Action must not be used: - for the correction of superficial wrinkles. - for injections other than intradermal - in combination with peeling, laser treatment or ultrasound-based - if the patient has cutaneous disorders, inflammation or infection at the treatment site or near to this site. - in the case of patients having a known hypersensitivity to hyaluronic acid, with a history of severe allergy or anaphylactic shock, - in case of patients with autoimmune diseases, - Due to possible interactions with other filling implants, which have not been researched, it is inadvisable to inject Teosyal 30G Global Action into sites in the presence of other filling implants. - in pregnancy, breast-feeding mothers, or in children - Do not inject into blood vessels - Do not inject into the eyelids

Posology and method of administration The injection of Teosyal 30G Global Action is reserved for practitioners trained in the injection techniques of products intended for the filling of wrinkles. Before beginning treatment, patients must be questioned regarding their past medical history and informed regarding the foreseeable outcome of treatment and of potential undesirable effects. Correction sites must be properly disinfected. It is advised to perform injections using the needle provided in the box. Teosyal 30G Global Action should be injected slowly into the mid-dermal layer using the linear threading injection technique. If the injection is made too deeply, i.e., into subcutaneous tissue, the correction will not have the sought after duration of action. An injection which is too deep, will encounter subcutaneous tissue which will not give rise to resistance during injection of the product, in contrast to the dermis. If the colour of the needle is visible under the skin during injection, this indicates that the injection will be too superficial, which it is also advisable to avoid, as the result of the correction might be irregular. The injection volume is dependent upon the correction required. It is however recommended not to exceed 3 ml per session. It is important not to overcorrect. Massage the treated sites carefully in order to ensure a uniform distribution of the product at the corrected sites.

Side effects: The practitioner must inform the patient that there are potential side effects related to the implantation of this device, occurring immediately following injection or after a delay. These include, among others: - Local manifestations: inflammatory reactions (erythema, oedema, pain at the point of injection), haematoma, itching, temporary loss of sensitivity around the injected area, dyschromia, abscesses, indurations, nodules (possibly granulomas), skin necrosis, migration of the implant. - General manifestations: immediate hypersensitivity up to anaphylactic shock. The patient should inform the practitioner of any secondary effect other than those described above or any effect persisting beyond one week. The practitioner will, in turn, inform the product retailer within the shortest time possible.

Assembly the needle on the syringe

For optimum use of Teosyal® products, it is important for the needle to be assembled on the syringe following the 4 steps detailed in the diagrams (see figures 1 to 5).

Precautions for use

Verify the expiry date and the integrity of the packaging before use. Do not use if the expiry date is exceeded or if the packaging is damaged. You should warn the patient against taking high-dose Vitamin E, aspirin, anti-inflammatories or anti-coagulants, during the week before the injections session. Advise the patient not to use make-up during the 12 hours following injection and to avoid extreme temperatures (intense cold, sauna,...) during the week following treatment. At the end of the treatment session, it is essential to discard all remaining unused product. This device is intended for single use only. Sterility is not guaranteed in the event of re-use. The gel can deteriorate, reducing its efficiency and the gel can dry out, hindering its extrusion through a needle. Dispose of used needles in appropriate containers.

Storage conditions

Store between 2°C and 25°C, away from direct sunlight. Make sure there are no visible signs of damage to the packaging before use.

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